

Mix all tubes immediately after draw with 7-10 gentle inversions.

802.747.1771

**LABORATORY- SPECIMEN TUBE TYPE**

**BLOOD BANK**

\*These specimens for Blood Bank require special training and can only be collected by trained RRMCC staff. **Please call Blood Bank before collecting at ext. 1809**

- \*ABO/Rh
- \*Antibody Screen
- \*Blood Bank RBC, PLT, FFP, CRYO, RHOGAM
- \*DAT
- \*Hold Clot
- \*Type and Screen



4 mL                      2 mL  
**Grenier or BD Lavender EDTA**  
4mL tube requires 2mL minimum  
2mL tube requires 1mL minimum

ABORH Newborn, DAT, Dir. Coombs

**COAGULATION**



3 or 2.7 mL                      2 or 1.8 mL  
**Grenier or BD Light Blue Sodium Citrate**  
-Grenier **MUST** fill to within triangle marker

-BD must fill to top of label

- D-Dimer
- DIC (add Lavender)
- Anti Xa (draw at RRMCC only)
- Fibrinogen
- Protime (PT/INR)
- PTT
- Thrombin Time

**HEMATOLOGY**



4 mL                      2mL  
**Grenier or BD Lavender EDTA**  
4mL tube requires 2mL minimum  
2mL tube requires 1mL minimum

- CBC
- DIC (+ Blue Top)
- H&H
- Immature Platelet Fraction
- Monoscreen
- Platelet Count
- Retic Count
- Sed Rate (ESR)
- WBC Count

**PLEASE SEE LABORATORY TEST CATALOG <https://rrmclab.testcatalog.org/> FOR ADDITIONAL ACCEPTABLE TUBE TYPES**

**CHEMISTRY**



3 or 5 mL **Grenier or BD Green/Yellow or Light Green Gel Separator**

- Acetaminophen
- Acetone
- Albumin
- Alkaline Phosphatase
- ALT
- Amylase
- AST
- Bilirubin Direct or Total
- BMP (*fasting recommended*)
- BUN
- C3 & C4 Complement
- CA 125
- Calcium
- Carbamazepine (Tegretol)
- Carcinoembryonic Ab (CEA)
- Cholesterol (HDL, LDL Total)
- CK
- CMP (*fasting recommended*)
- Creatinine
- C Reactive Protein
- Digoxin
- Dilantin (Phenytoin)
- Ferritin
- FSH
- Gentamicin
- GGT
- Glucose (*fasting recommended*)
- HCG Quantitative
- Haptoglobin
- Hepatic Function Panel
- High Sensitivity CRP
- High Sensitivity Troponin
- Iron/TIBC
- LDH **[keep Room Temp]**
- LH
- Lipase
- Lipid Profile (*fasting recommended*)
- Lytes
- Magnesium
- NT-proBNP
- Phenobarbital
- Phosphorus
- Prolactin
- Procalcitonin
- PSA
- Renal Panel (*fasting recommended*)
- Salicylate
- T3 Free & T4 Free
- T4 Total
- Total Protein
- Triglycerides
- TSH
- TSH Cascade
- Uric Acid
- Valproic acid
- Vancomycin
- Vitamin D Total (25 hydroxy)
- VB12

**CHEMISTRY (cont.)**



4 mL                      2 mL  
**Grenier or BD Lavender EDTA**  
4mL tube requires 2mL minimum  
2mL tube requires 1mL minimum

Ammonia (deliver to lab on ice within 15 minutes)  
HGB A1C



5 mL  
**Grenier or BD Gold Gel Separator**

- CCP Abs
- Cortisol
- Estradiol
- Folate
- Hepatitis Acute Panel (2 tubes)
- HIV Ab 1/2 & p24 Ag
- Lithium
- Lyme Serology or Western Blot
- Mumps Ab
- Rubella Ab
- Rubeola (Measles) Ab
- Progesterone
- Syphilis Ab
- T3 Total
- Varicella zoster Ab



4 mL  
**Grenier or BD Dark Green Lithium Heparin**  
**ON ICE, STAT TO LAB WITHIN 15 MINUTES**

- Carboxyhemoglobin
- Critical Care Panel (CCP)
- Ionized Calcium
- (included in CCP)
- Lactate
- (included in CCP)
- Arterial Blood Gas
- (included in CCP)
- Venous Blood Gas
- (included in CCP)

**REFERENCE (Send-outs)**

Send-out testing may require special handling procedures and specific volumes. Please call ext. 1794 for confirmation.

**LABORATORY- SPECIMEN TUBE TYPE**



Aldolase  
Aldosterone  
Angiotensin Converting Enzyme  
Beta 2 Microglobulin  
C1 Esterase Inhibitor Functional  
CA 15-3  
Calcitonin Level  
Clozapine (Clozaril)  
Complement Total  
Lipoprotein  
Phenytoin T&F  
Serotonin  
Trazodone Level



Acetylcholine Receptor Binding Ab  
Alkaline Phosphatase Isoenzymes  
Alpha 1 Antitrypsin  
Antineutrophil Cytoplasmic Ab ANCA  
Beta-Carotene (**light protect**)  
CA 19-9  
CA 27-29  
Cardiolipin Ab  
Celiac Disease Serology Cascade  
Celiac Disease Comprehensive Cascade  
Ceruloplasmin  
Cytomegalovirus CMV IgG  
Epstein Barr Virus Ab Panel  
Erythropoietin  
Herpes Simplex Virus Ab  
Hypersensitivity Pneumo Panel IgG  
IgE  
IgG Subclasses  
Immunoglobulin Free Light Chains  
Insulin  
Ionized Calcium (**DO NOT OPEN**)  
Keppra (Levetiracetam)  
Lamotrigine (Lamictal)  
Mitochondrial Ab  
Osmolality  
Prealbumin  
Protein Electrophoresis  
Thyroglobulin Tumor Marker  
Thyroid Abs  
Thyroperoxidase  
TICK ANTIBODY Panel (NOT DNA)  
Tobramycin  
Transferrin



**-Grenier MUST fill to within triangle marker**  
**-BD must fill to top of label**  
**[Call for special handling]**  
AT3 Functional (2 Tubes)  
Factor 2 (II) (2 Tubes)  
Factor 8 (VIII) (2 Tubes)  
Factor 9 (IX) (2 Tubes)  
Factor 11 (XI) (2 Tubes)  
Factor 12 (XII) (2 Tubes)  
Lupus Anticoagulant Profile (7 Tubes)  
Protein C&S (2 Tubes)



ACTH [draw at RRMC only]  
CD4 T-Cell [Room Temp]  
Cyclosporine  
FK506 (Tacrolimus)  
Hemochromatosis HFE [Room Temp]  
HIV1 Quantitative (2 Tubes)  
HLA B27 [Room Temp]  
Homocysteine [Spin, Sep, & Freeze]  
JAK2 [Room Temp]  
PTH  
Lead  
Sirolimus (Rapa)  
TICK Borne DNA Panel  
Vitamin B1 (Thiamine) **Overnight Fast, light protect whole blood in amber vial and freeze immediately.**



Celiac Comprehensive Cascade  
Cystic Fibrosis Carrier  
Factor 5 (V) Leiden Mutation  
MethyleneTetraHydroFolate Red Mut  
PT G20210A Mutation  
Tay Sachs (2 full tubes)

Order of draw for multiple tube collections and all departments:

Blood Cultures – **Blue - Gold - Red - Royal Blue MONOJECT - Green – Royal Blue MONOJECT - Lavender - Yellow ACD**  
**FIRST** **LAST**