




## Collection Swab Types for Chlamydia, Gonorrhea, Mycoplasma genitalium (Mgen), Trichomonas vaginalis (Trich), and Vaginitis Testing

Sample Type	Swab Type	CT-NG Panel NAAT	Mycoplasma genitalium	Trichomonas vaginalis	Vaginitis Panel
ThinPrep Pap Test	ThinPrep Pap Vial	X		X	
Clinician Collected Vaginal Swab	Aptima Multitest Swab (orange tube) 	X	X	X	X
Clinician Collected Throat Swab		X			
Clinician Collected Rectal Swab		X			
Patient Collected Meatal Swab			X		
Patient Collected Vaginal Swab		X	X	X	X
Clinician Collected Endocervical	Aptima Unisex Swab (blue swab) 	X	X	X	
Clinician Collected Male Urethral Swab		X	X		
Female Urine	Approx 30 mL of voided, first catch, dirty urine. The "dirty thirty" 	X	X	X	
Male Urine		X	X	X	

- **Clinician Collected Endocervical/Male Urethral Swabs must be collected using the Aptima Unisex Swab (blue swab).** Specimen will be rejected if collected with invalid swab type
- **Clinician Collected Swabs must be done with ordering provider. Please write source on swab or specimen label**
- Patient-collected vaginal and meatal swab specimens must be collected in a clinical setting, such as a medical office or a laboratory blood draw station.

Any questions on collection please call 802-747-1801 (cytology) M-Fri 9 am- 4 pm or 802-747-1771 (lab main line)

Testing schedule: CT-NG, Vaginitis Panel M-F 9:30 am- 4:40 pm, Mgen tbd.

Trich vaginalis is a send out through Mayo as order Misc Mayo and comment GOKEY TVRNA